253 | Neolution®Air Loop | Gebrauchsanweisung / Instructions for Use

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ENGLISH: Filtering half masks - INSTRUCTIONS FOR USE


IMPORTANT: The Neolution® Air Loop filtering half masks are Personal Protective Equipment Category III in compliance with Regulation (EU) 2016/425 against solid and liquid aerosols. Type tested according to EN 149:2001+A1:2009. Protection category FFP2 or FFP3 as marked on the mask.

APPLICATIONS: FFP2: Filtering half masks of category FFP2 protect against solid and liquid particles of all fine dusts up to 10 times WEL, [i.e. APF=10]. FFP3: Filtering half masks of category FFP3 protect against solid and liquid particles of all fine dusts up to 20 times WEL, [i.e. APF=20].

CHECK BEFORE USE: Before use, make sure that the filtering half mask, including all components, has no holes, cracks or other damage. If you discover any damage, DO NOT USE THE MASK. Use a new one. National regulations must be observed, e.g. DGVV regulation 112-190 (BGR 190) "Use of respiratory protective equipment", BGI 504-26 "Selection criteria for special occupational medical precautions in accordance with the employers' liability insurance association principle, G26 Respiratory protective equipment".


FITTING INSTRUCTIONS: **1.** Separate top and bottom panels at the back of the mask to form a cup shape. **2.** Ensure both panels are fully unfolded. **3.** Cup the bottom panel under your chin and pull both elastics behind your ears. **4.** Adjust top and bottom panels for a comfortable fit and ensure panels are not folded inwards. **5.** Using both hands, press nose clip around nose to the shape of the nose for a good facial seal. **6.** To test the fit, cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the ear loops for better fit. *NOTE: Change respirator immediately if breathing becomes difficult or respirator becomes damaged or deformed in such a way that an optimal seal can no longer be guaranteed. Use mask only in accordance with these fitting instructions.

253 | Neolution® Air Loop | Gebrauchsanweisung / Instructions for Use

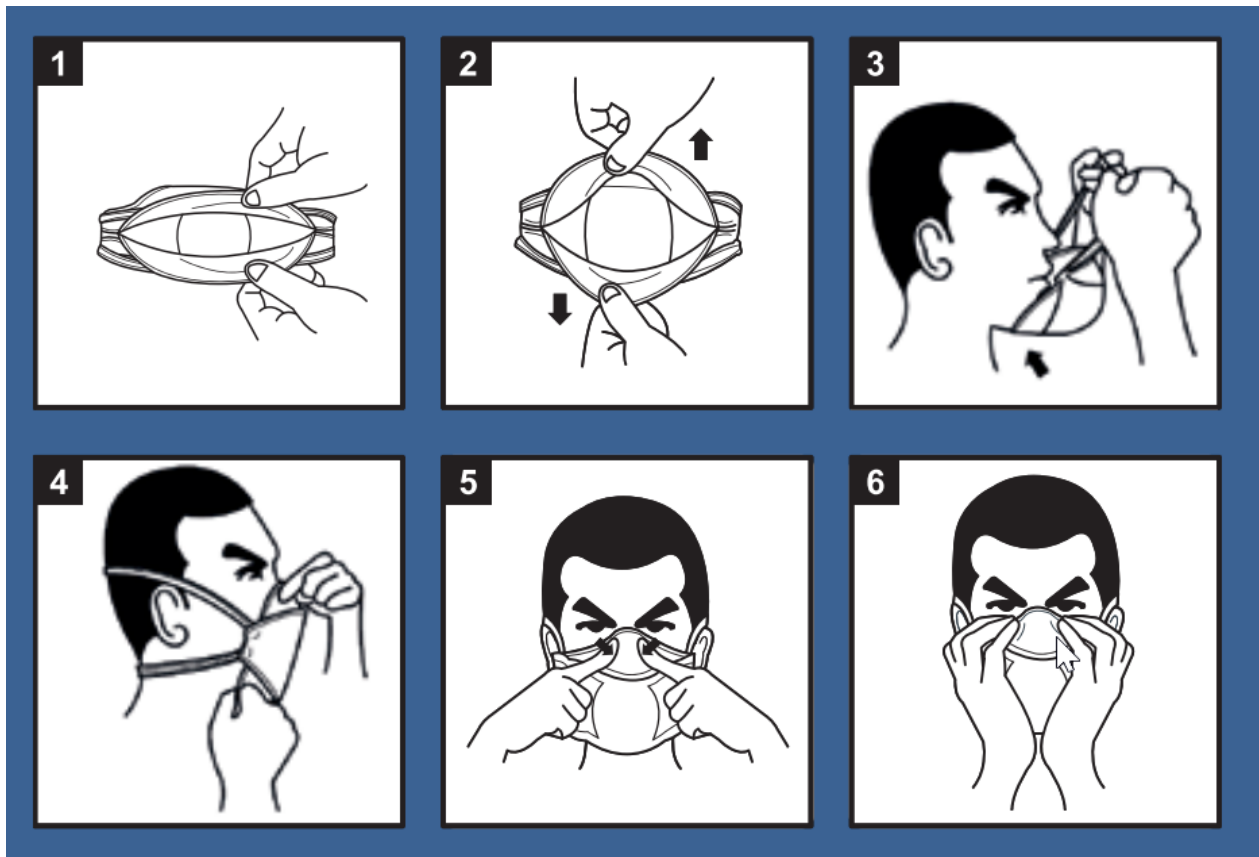WARNINGS: **1.** Failure to follow the instructions given here or restrictions on use of this product may reduce the effectiveness of this mask and result in nausea or even death. **2.** A carefully selected mask is essential to protect your health. Before using this mask, consult an appropriate safety organisation to determine if this mask meets the requirements of the intended use. **3.** Filtering half masks are dependent on the ambient atmosphere. Use only in adequately ventilated environments. Do not use if the atmosphere contains less than 19 % oxygen. **4.** Respirators must not be used if the concentration of harmful substances poses an immediate threat to health or life. **5.** Do not use in an explosive atmosphere. **6.** Leave the work area immediately, (a) if breathing becomes difficult, (b) dizziness or exhaustion occurs. **7.** Facial hair and special characteristics of the face shape may reduce the effectiveness of this mask. **8.** Do not make any changes or modifications to the mask. **9.** Intended for single use. No maintenance required. Dispose of properly after use. **10.** Store masks in packaging away from direct sunlight until ready to use.

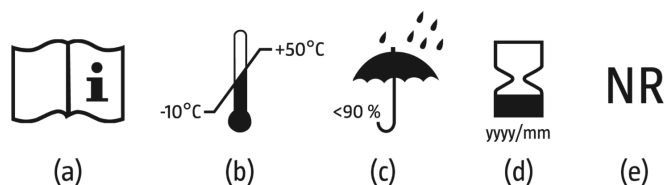MEANING OF THE ICONS/SYMBOLS: (a) See user information. (b) Temperature range during storage. (c) Maximum humidity level during storage. (d) End of shelf life. yyyy = year; mm = month. (e) NR: suitable for single use only (maximum wearing time of 8 hours). Do not reuse.

Marking and performance recorded in technical tests applicable to this PPE:

neolution Air Loop
REF 253 | FFP2 NR | EN 149:2001+A1:2009 YYYY/MM **CE0161**

253 | Neolution® Air Loop | Gebrauchsanweisung / Instructions for Use

Meaning marked: manufacturer's brand -

PPE reference | Classification: FFP2 NR | Applicable Standard - expiration date - CE-marking + Notified Body of production control: CE 0161

Levels or classes of protection offered by PPE intended for respiratory protection: (1) Maximum penetration of the filter material (Paraffin Aerosols and Sodium Chloride) (2) Breathing resistance (3) Inward Leakage

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